



The Red Flash


News from our laparoscopic products

Dear valued Customer,

we are pleased to present you an innovative modification in our laparoscopic program:

Multifunctional Plastic Handle For Laparoscopic Instruments

As per now we will deliver the following instrument for article numbers H43-...

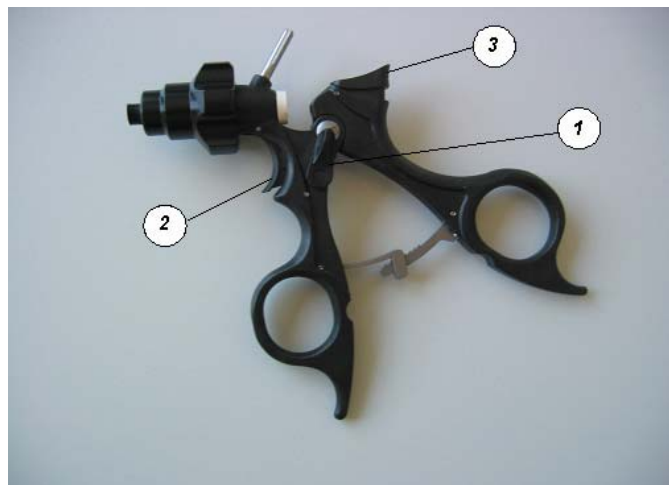

With this new plastic handle it is possible to work with or without ratchet due to the fact that the ratchet can be deactivated . The deactivation and renewed activation will be effected by a lever (1) on the side of the instrument.

There are two possibilities to loosen the active ratchet: Either by pushing the release button (2) at the front part of the handle (e.g. with the index finger) or by pushing (e.g. with the thumb) an additional release button (3) on the upper part of the handle.



The Red Flash

For the plastic handle with article numbers H41... we will from now on deliver the following model:



The plastic handle without ratchet has been modified and improved regarding its form, too. It now is perfectly ergonomic and facilitates the surgeons' work.

Short term delivery will be possible for all handles.

Please contact our customer service for further information at any time.

We appreciate your interest in our products and thank you very much for the confidence shown to us.

The Hermann Team